

# XSTAT<sup>®</sup> 12

## **ATTN TREATMENT FACILITY:**

**A hemostatic dressing for non-compressible hemorrhage was used in the treatment of this casualty. Each device comprises approximately 38 small hemostatic sponges individually marked with an X-shaped radiopaque marker.**

### **PRODUCT REMOVAL INSTRUCTIONS:**

**WARNING:** Sponges must be removed intraoperatively by surgeon with the capability and equipment for achieving proximal and distal vascular control.

- 1) Survey the wound site and assess potential vascular bleeding sites and develop plan to achieve surgical control of injured vessel(s).
- 2) Remove sponges from the wound site manually and/or with surgical forceps to the site(s) of bleeding.
- 3) Thoroughly explore wound and remove all sponges.
- 4) Prior to wound closure, obtain plane x-ray, optimally in more than one projection. The presence of retained sponges may be easily missed on radiographic images. Thoroughly examine x-ray for radiopaque x-pattern of sponges that may be inadvertently retained in the wound cavity.
- 5) If sponges are identified via x-ray, carefully re-examine wound cavity and remove them. Perform and review second x-ray to confirm complete sponge removal.
- 6) The XSTAT elicited a mild pyrogenic response in biocompatibility tests. Monitor patient for rise in temperature, chills, hypotension, and septic shock.



**Contains material derived from shellfish.**

3-Pack Part Number: FIN-0001-02  
1-Pack Part Number: FIN-0001-04

**REVMED** X<sup>™</sup>

RevMedx, Inc.  
25999 SW Canyon Creek Road Suite C  
Wilsonville, OR 97070  
PH: 503.218.2172 [www.RevMedx.com](http://www.RevMedx.com)  
Label ID: RAW-0057-05  
Label Date Issuance: 2016-09-06

